

General Information

Reusable earplugs made of impregnated sheep's wool.

Material: Impregnated sheep's wool

Colour: Skin-coloured

Characteristics: The wool warms the ears in wind and cold, keeps them dry in wet conditions and protects against water splashes when showering, washing the hair and bathing. The hearing ability and the normal ventilation of the ears are not impaired. The product is resistant to salt water and also suitable for children.

Instructions: IN-Ear plug (to be inserted into the ear canal)



Insert the earplug tightly into the ear canal and check occasionally for tight fit.

For tympanic tubes, chronic ear problems and injuries to the eardrum, please consult your doctor before use.

Further Information

- Ensure hygienic use. Your hands, ears and earplugs should be clean and dry!
- OHROPAX® Windwolle can be reused as long as it is hygienically perfect. After use, carefully squeeze out the plugs to remove residual moisture and then dry well in the air. Cleaning with soap is not possible as this may remove the water-repellent impregnation.
- Keep away from small children (danger of swallowing).
- Can be used by children aged six and older; use only under adult supervision.
- Warning: The product does not provide adequate protection when swimming and diving
- Storage: In the box, dry.

Application Areas

For private use: OHROPAX® Windwolle is ideally suited for wind and cold due to its high wearing comfort, keeps it dry in wet conditions and protects against water splashes during showering, hair washing and bathing.

Tolerability

OHROPAX® Windwolle is considered to be extremely well tolerated. Components with known allergenic potential are not included.